Score[®]

experts in seating

score balance

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 Before using the chair, consult the manual


components



Gaslift


Base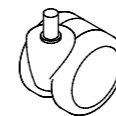

Castors
5x

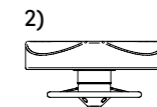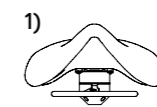Seat
1) Saddle
2) Medical Line

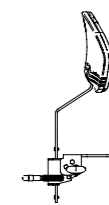
Lumbar support


Locking pins
2x

dimensions


score amazone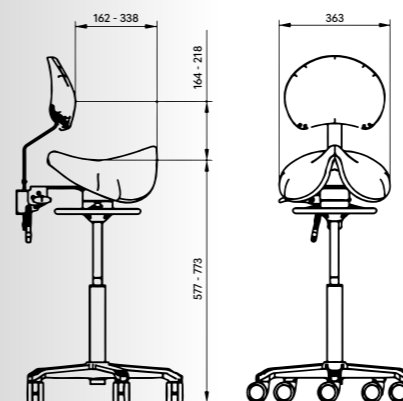

score jumper

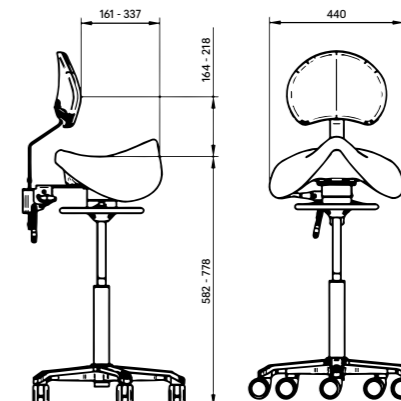score medical 6100 / medical 6300

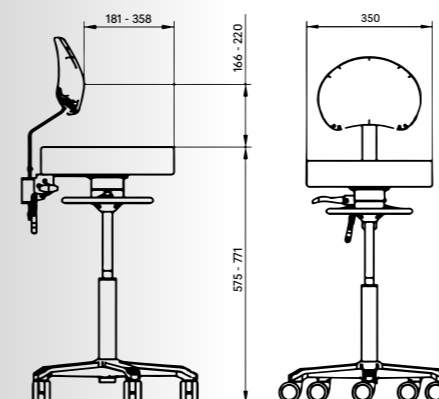Seat height range


Heights mentioned in the drawing are based on the standard model with gas spring MID31 (20 cm)

Gaslift low

adjustment range MID25: 15 cm

Gaslift high (Jumper and Amazone saddle chairs)
adjustment range MID38: 25 cm


mounting

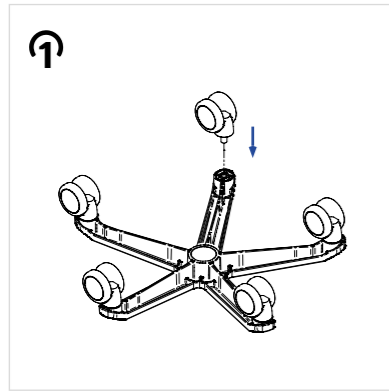
1 Insert the castors into the base


2 Remove the cap from the gaslift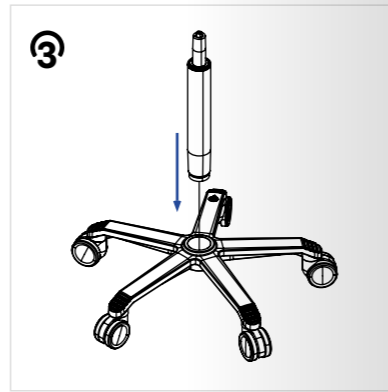


3 Insert the gaslift into the base

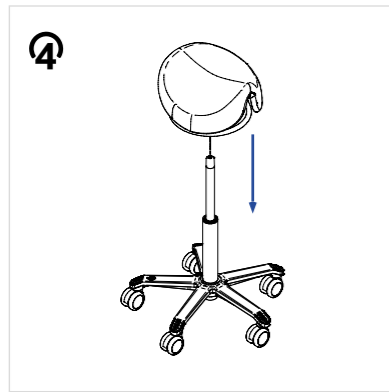
4 Press the seat onto the gaslift

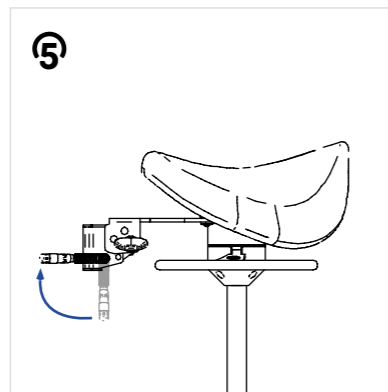
5 Turn the lumbar support handle up a quarter turn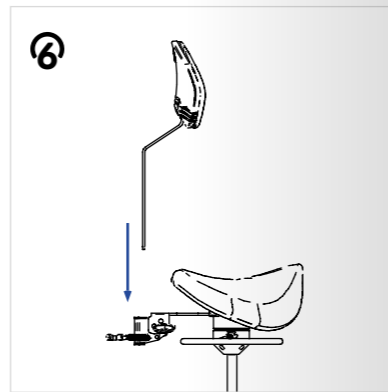

6 Place the lumbar support in the mechanism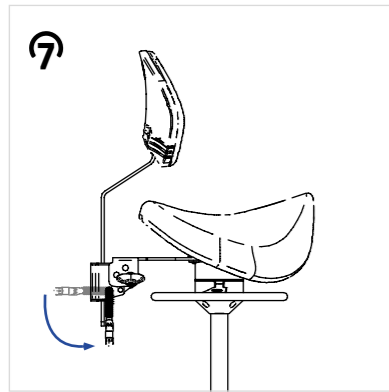

7 Turn the lumbar support handle back down and lock the height position of the lumbar support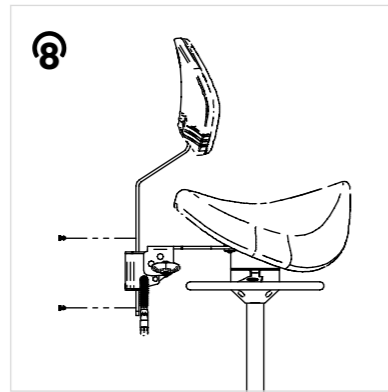

8 Insert the locking pins into the back bar

cleaning

Cleaning of surfaces, chrome-plated and coated parts and castors

Clean the parts with a damp cloth, possibly with a mild detergent without abrasives. After using a damp cloth, dry parts with a dry cloth. The thinner tube on the gaspring is equipped with grease and a grease scraper. This tube keeps itself clean. The greasy part should never be wiped with water or any other detergent.

Cleansing method Stamskin Top

Wipe Stamskin Top daily with a damp lukewarm/warm cloth. If necessary, use a mild soap (Ph neutral, free of acidic or alkaline excipients). Do not use alcohol, chlorine, ammonia or acid-based cleaning products during regular maintenance. Also, do not use solvents or abrasives. After cleaning, dry the upholstery with a dry cloth.

For general stains, we recommend Stamskin Cleaner. Using a soft brush, you can apply Stamskin Cleaner to the surface to be treated with circular movements. Leave the Stamskin Cleaner on for 3 minutes. Then use a dry cloth to remove the Stamskin Cleaner along with the dirt.

If blood or other bodily substances come into contact with the Stamskin Top upholstery, disinfect it immediately with a cold solution of 10,000 ppm chlorine and water. After wiping with a clean wet cloth, wipe the area and then dry with a clean dry cloth.

A list of tested cleaning products is available for Stamskin Top. If you have any doubts about your cleaning product, please contact the Score supplier.

Cleaning method PUxx

Wipe PUxx daily with a damp lukewarm/warm cloth, use a cleaning agent if necessary. PUxx is resistant to most cleaning and disinfecting agents based on 70% alcohol, chlorine or ammonia. Follow the instructions of the disinfectant manufacturer. Always wipe the cleaning and disinfecting products with a clean

damp cloth. Do not use solvents or abrasives. After cleaning, dry the upholstery with a dry cloth.

Occasionally wipe the PUxx parts that come into contact with the skin (e.g. upholstered armrests) with sufficient warm water in combination with Nirocleaner or another alkaline cleaning agent to remove the remaining soap and skin oils. Use a brush if necessary. It is best to leave these cleaning products on for 3 minutes before removing the detergent and dirt with a cloth with clean water. By properly removing the cleaning product, contamination is less likely to take place.

Remove blood stains immediately with a cold disinfectant. After disinfecting, dry the upholstery with a dry cloth.

Stamskin Top properties

Multi-layered artificial leather PVC on a polyamide-jersey base cloth. Resistant to urine, blood, sweat, mould, and water. Abrasion resistance > 120,000 Martindale (EN ISO 5470-2). Fire-resistance: M2/NFP92.507, Test cigarette / Match EN 1021.1-1021.2, IMO MSC.307 (88) (201 FTP Code) Annex 1 part 8, Aeronautical test FAR/JAR 28-853/855.- Fungicidal: Degree 0, excellent (EN ISO 846-A). Sanitized® hygiene function: protection against mould and algae formation. Biocides: Stamskin Top contains Folpet (N°528/2012/CE-Folpet), a fungicidal additive that is used to preserve the exterior and physical properties of these items, and to extend their service life. Folpet is a phthalate (plasticizer). Patient chair with PUxx upholstery is free from phthalates/plasticizers

PUxx properties

- 100% polyurethane on polyester/cotton backing. Completely free from PVC and phthalates.
- 100% waterproof, resistant to urine and blood.
- Abrasion resistance 300,000 Martindale (EN ISO 5470-2).

- Fire-resistance: UK BS5852 Crib 5, EN 1021 1-2, CALL117-2013, IMO 2010 FTP: part 8.
- Light-fastness 6-7 (8 is very good).
- Can be disinfected using alcohol, chlorine, or hydrogen peroxide solutions.
- Oeko-Tex certified.

maintenance

Periodically check the product for function and for damage. Periodic checking contributes to a longer service life and trouble-free use. Reduced functionality may indicate future problems. Is a component damaged or is its operation impaired? If so, please contact your Score supplier.

The Score saddle or the Score stool has been designed and tested for a normal service life of ten years. The warranty on the product is five years.

Warranty: Only applicable to products in original condition. Warranty only applies to a technical malfunction/defect in one or more parts and manufacturing defects in the upholstery/upholstery method.

No warranty: signs of wear occur due to daily use of the product.

Transport damage, returns and delivery conditions


For transport damage, returns, defects and delivery conditions, please contact your Score dealer.

- **The product has an undamaged and original product sticker. This product sticker must not be removed from the chair**
- **Keep this manual in a safe place**
- **Do not use the chair outdoors**
- **Do not roll the chair over cables and/or cords**
- **Do not stand on the chair**
- **Do not use the chair when it has defects**
- **Do not use the chair when the upholstery is damaged**
- **Report defects immediately to your Score supplier. To identify the product, always mention the calc. number indicated on the product sticker. If possible, send photos of the product sticker, and of the defect or damage.**

score, then you're in the right place!

The Score Amazone and Jumper saddle chair and the Score Medical Line 6211 and 6311 stool are a work stool for professional use. Please read this manual before use. You should be aware of the working protocols of the environment in which this product is placed. This work stool is not intended or intended for persons with injury, disability or physical illness. If in doubt, contact your supplier.

adjustments

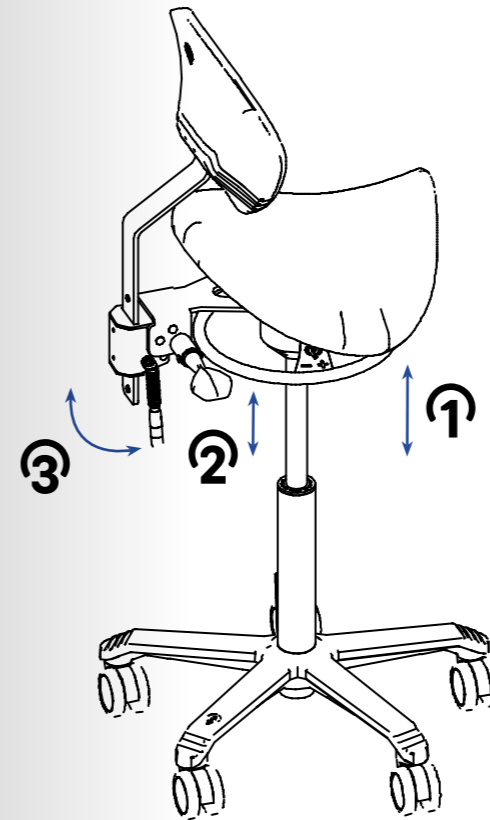
- 1 Pull the ring upwards to operate the gaslift and adjust the seat height. Rotate the ring to the left or right to change the degree of movement of the seat.
- 2 Pull the front lever upwards to adjust the lumbar support forwards or backwards.
- 3 Pull the rear lever back to slide the lumbar support up or down.

Scan to watch the instruction video here

instructions for use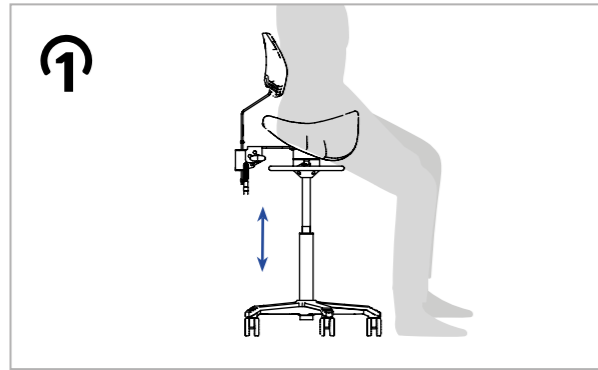


Score Amazone/ Jumper


Use the ring to adjust the height of the chair to a 115 to 135-degree knee angle.

Score Medical 6211/6311


Use the ring to adjust the height of the chair to a 100 to 115-degree knee angle.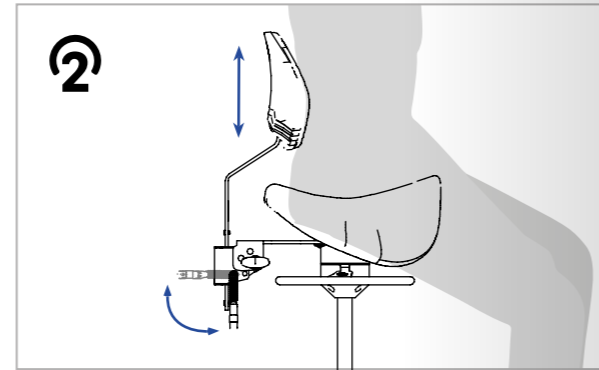

Adjust the height of the lumbar support by turning the rear lever up a notch. Choose the height of the lumbar support by placing it in the hollow part of your back. Lock the lumbar support by turning the handle downwards.

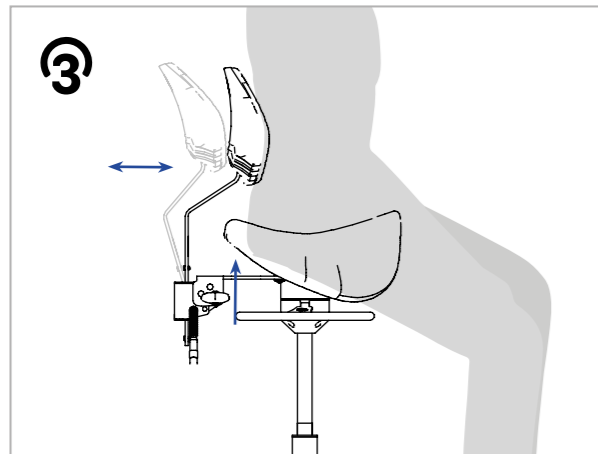Pull the lever up and choose the support you want in your back with a little more or less pressure. Release the lever to lock the position of the lumbar support.

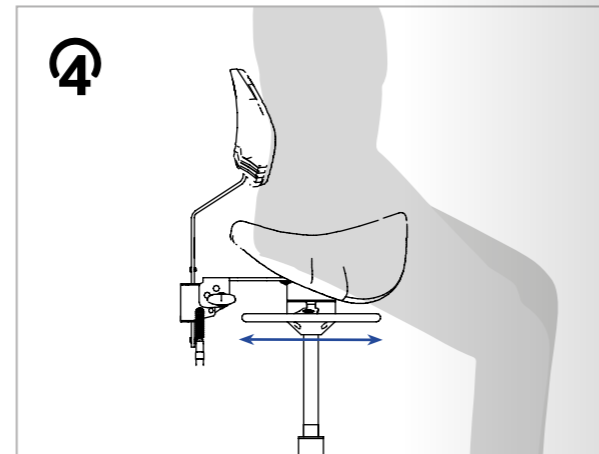The degree of movement of the Balance mechanism can be adjusted by rotating the ring; Turn the ring into the direction of the + for a more fixed seat, direction of the - for a more flexible seat.

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