## Vena<sup>®</sup> Hydro phlebotomy chair

NLD	Bloedafnamestoel	- Gebruiksaanwijzing
ENG	Phlebotomy chair	- User Manual
DEU	Blutentnahmestuhl	- Gebrauchsanweisung
FRA	Fauteuil de prélèvement	- Mode d'emploi
ITA	Poltrona per prelievi	- Manuele d'uso

**Fabrikant:**

Score B.V.  
Feithspark 5  
9356 BX Tolbert  
Nederland  
Tel: +31 (0)594 554 000  
E-mail: [info@scoreseating.com](mailto:info@scoreseating.com)  
Website: [www.scoreseating.com](http://www.scoreseating.com)

**© Copyright 2023 Score B.V.**

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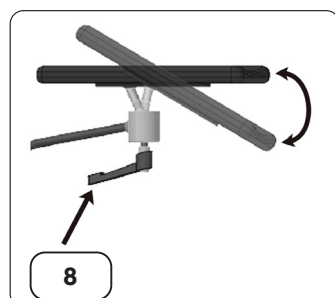
**© Copyright 2023 Score B.V**

All rights reserved. It is not permitted to copy, amend, or (partially) duplicate the contents of this user manual for any purpose without express written permission from Score B.V. Text and images are protected by copyright. This user manual applies to a medical device. In the event of incorrect use or misuse of this user manual, please contact Score B.V.

Vena®, Score® and the Score logo are (registered) trademarks of Score B.V.

The information from this user manual may be amended at any time without prior notification. In case of doubt, contact your supplier or the manufacturer for the most up-to-date information.

## Content

1.	General information .....	22
1.1	Product description and intended use .....	22
1.2	Intended patient population .....	22
1.3	Intended user environment and user population.....	22
1.4	Critical performance.....	22
1.5	Supplier declaration.....	22
1.6	All users please note: .....	22
1.7	Warnings and points for attention .....	22
1.8	Harmful substances.....	23
1.9	Medical device identification label.....	23
1.9.1	Patient chair identification.....	23
1.9.2	Armrest identification:.....	24
2.	Product Description.....	25
2.1	Options and accessories .....	25
3.	Vena <sup>®</sup> Hydro installation and first use.....	26
3.1	Unpacking the product .....	26
3.2	Installation of the product.....	26
4.	Instructions for use .....	27
4.1	Preparing for treatment.....	27
4.2	Adjusting the angle of the backrest and legrest.....	28
4.3	Getting in and out of the Vena <sup>®</sup> Hydro .....	28
4.4	Operating the clamping lever .....	29
4.5	Adjusting the Armrest .....	29
4.6	Foot pedal for height adjustment and rotation lock.....	30
4.7	Fainting of the patient .....	30
4.8	Safety measures for the Vena <sup>®</sup> Hydro.....	30
5.	Instructions for daily maintenance and cleaning.....	31
5.1	Cleaning PU surfaces, chrome-plated and coated components, feet/castors.....	31
5.2	Cleaning the upholstery.....	31
5.2.1	Stamskin Top upholstery: properties and cleaning method .....	32
5.2.2	PUxx upholstery: properties and cleaning method .....	33
5.3	Cleaning after a blood squirt incident.....	33
6.	Maintenance .....	33
6.1	Maintenance interval .....	33
6.2	Regulatory checks .....	34
7.	Dimensions Vena Hydro .....	35
8.	Complaints and safety notices .....	35
8.1	Product safety and functionality notifications. ....	35
8.2	Complaints.....	35
8.3	Contact details .....	35
9.	Quick guide .....	35

## 1. General information

Please read the instructions in the following chapters of this manual carefully. This user manual was developed specifically for the Vena® Hydro phlebotomy chair from Score. The Vena Hydro has been developed, tested, manufactured and marketed in accordance with current legislation and guidelines. When using the medical device, the user must be familiar with the contents of this manual.

To ensure a safe environment during the blood sampling process, it is important that you are aware of the applicable guidelines and protocols imposed by the (care) institution where this Vena Hydro is used. In addition, you must be familiar with the instructions for use of possible additional medical devices used during the blood sampling process.

Store this user manual carefully near the medical device. In the event of questions or comments, please contact the supplier of this medical device or the manufacturer as listed on the back page of this user manual.

### 1.1 Product description and intended use

The Vena Hydro phlebotomy chair is a medical device that is intended to support the body and position the patient's arm during blood collection, blood donation, plasma donation or administration of an injection.

The Vena Hydro is a height-adjustable patient chair equipped with two phlebotomy armrests. The backrest of the chair can be adjusted from sitting to a semi-reclining position. The back angle can be adjusted up to 30° backwards.

The Vena Hydro is a non-invasive, multiple-use medical device with a usage time of up to 60 minutes.

### 1.2 Intended patient population

The Vena Hydro was designed as a patient chair for adults with a maximum body weight of 150 kg. Children from 8 years of age can be treated in the chair as long as they are supported or accompanied by an adult.

**Warning:** suitable for persons weighing up to 150 kg

### 1.3 Intended user environment and user population

The Vena Hydro is a non-sterile product. The Vena Hydro is intended for use in a professional medical environment where blood is collected, donated, or where injections are administered for medical purposes. This medical device must be operated by a physician or nurse who is trained to collect blood or administer injections.

### 1.4 Critical performance

The Vena Hydro phlebotomy chair is intended for physical support and correct positioning of the patient's arms during the blood collection process, donation process, or when administering an injection. The patient can be both seated upright and in a reclining position for treatment. The chair is designed so that it can be fitted with a blood collection armrest that supports the patient's arm for blood collection or injection. In the event the patient becomes unwell, they can be moved into the Trendelenburg position.

### 1.5 Supplier declaration

This product is supplied by Score B.V.'s distribution channel. For questions or comments about this medical device, please contact your Score supplier. If you prefer, you may also contact the manufacturer, Score B.V. Contact details are listed on the back page of this user manual.

### 1.6 All users please note:

Score B.V. would like to advise all users of the following before working with this medical device:

- > every user must have read the user manual and be aware of the functionalities of the chair, as well as possible risks
- > every user must be aware of the procedures and protocols in place at the location of the Vena Hydro medical user environment
- > every user must have been instructed by a qualified person.

### 1.7 Warnings and points for attention

If extra attention is required during use, or if a safety warning is applicable, this will be further identified in this user manual with the use of a symbol.

In this user manual, warnings are specified by '**Warning!**' Warnings are intended to indicate that use or incorrect use of the medical device may lead to injury, death, or other adverse consequences.

Attention points mentioning "**Caution**" are intended to point out incorrect use resulting in defect or damage to the medical device

The following warning or safety symbols are used in this manual.

	<p><b>WARNING or CAUTION: General symbol for attention/safety instruction</b>          Instructions, directions and/or precautions for safe use for the patient and/or the person operating the medical device are highlighted with this symbol.</p>
---	--

## 1.8 Harmful substances

This medical device and its accessories does not contain any harmful substances or other materials that can affect human health.

This medical device or the medical device's accessories do not contain animal-derived materials.

Biocompatibility: the Vena Hydro is a non-invasive medical device. The contact surfaces are kind to skin and free from harmful substances.

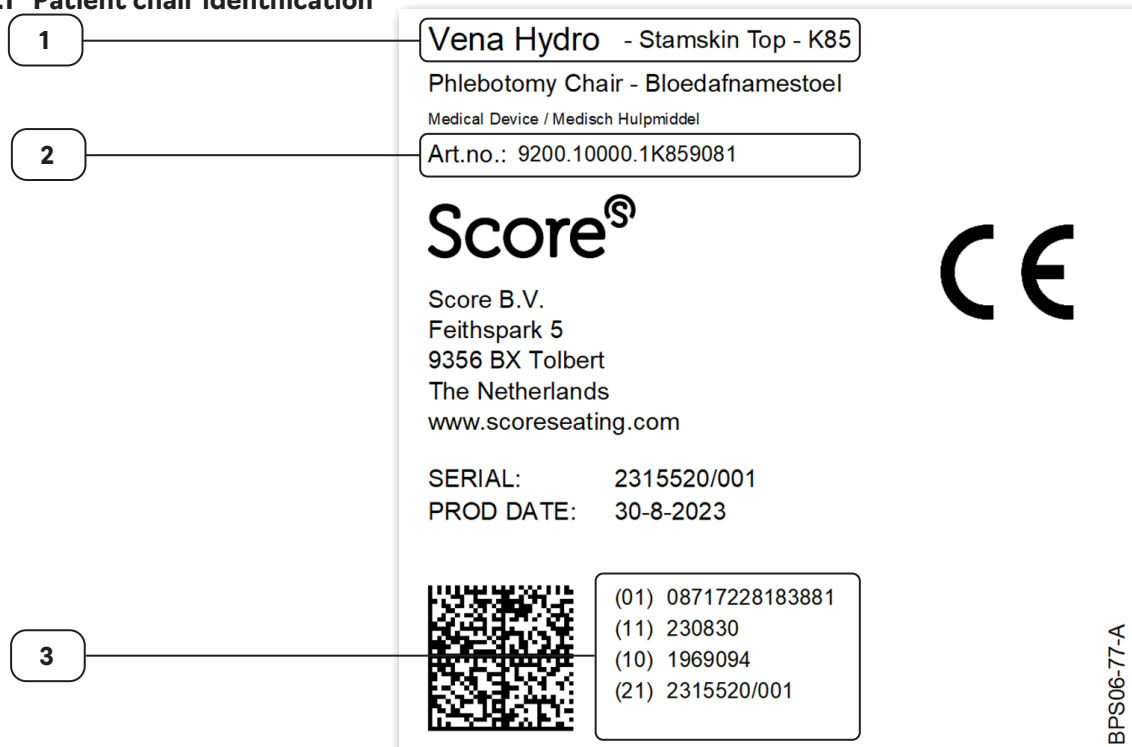
Biocides: Stamskin Top contains Folpet (N°528/2012/CE-Folpet), a fungicidal additive that is used to preserve the exterior and physical properties of these items, and to extend their service life. Folpet is a phthalate (plasticiser). Patient chair with PUxx upholstery is free from phthalates/plasticisers.

## 1.9 Medical device identification label

The Vena Hydro is a medical device and accordingly, it has a product ID. This label contains the Unique Device Identification (UDI) code that can be used to trace its production data. Do not remove this label. In the event of a service enquiry or a complaint, the UDI code must be used as a reference, as well as the identification data printed on the medical device's label. If the label is not present or illegible, please contact the manufacturer or supplier of the device immediately.

The Vena<sup>®</sup> Hydro patient chair is always fitted with two Vena<sup>®</sup> armrests. The armrests are interchangeable and come with their own identification labels.

### 1.9.1 Patient chair identification

**1** Vena Hydro - Stamskin Top - K85  
Phlebotomy Chair - Bloedafnamestoel  
Medical Device / Medisch Hulpmiddel

**2** Art.no.: 9200.10000.1K859081

**3**  (01) 08717228183881  
(11) 230830  
(10) 1969094  
(21) 2315520/001

Score<sup>®</sup>  
Score B.V.  
Feithspark 5  
9356 BX Tolbert  
The Netherlands  
www.scoreseating.com


CE

BPS06-77-A

Number	Explanation
1	This is where the product name is listed, followed by the type of upholstery and the corresponding Score upholstery colour code.
2	Item no: is the Score item number for the product. The first 9 digits (92xx.xxxxx) correspond with the UDI-DI number. The last part of the item number (.1K859081) is an alphanumerical field that denotes configurable aspects such as colour, legs/feet and armrest options.
3	Legal identification of the product (machine-readable): (01): Product UDI-DI (11): Product manufacturing date (10): BATCH number (21): Product serial number (unique)

	<p><b>CAUTION: Do not remove label</b></p> <p>The product label must be affixed to the product. Contact the supplier or manufacturer of the medical device immediately if there is no label with UDI code attached to the device.</p>
	<p><b>CAUTION: UDI-DI code</b></p> <p>In case of comments, service, or complaints related to the product, always refer to the UDI code of the medical device.</p>

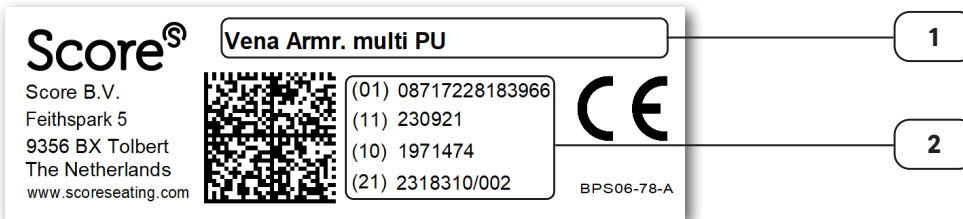**1.9.2 Armrest identification:**

Number	Explanation:
1	This is where the product name is listed, followed by the type of upholstery and the corresponding Score upholstery colour code.
2	Legal identification of the product (machine-readable): (01): Product UDI-DI (11): Product manufacturing date (10): BATCH number (21): Product serial number (unique)

	<p><b>CAUTION: Do not remove label</b></p> <p>The product label must be affixed to the product. Contact the supplier or manufacturer of the medical device immediately if there is no label with UDI code attached to the device.</p>
	<p><b>CAUTION: UDI-DI code</b></p> <p>In case of comments, service, or complaints related to the product, always refer to the UDI code of the medical device.</p>

## 2. Product Description

Overview of Vena Hydro patient chair and interface:

ENG

Number	Explanation:
1	Backrest
2	Seat
3	Foot pedal for height adjustment Vena Hydro
4	Armrest for blood collection
5	Operating lever for adjusting the backrest inclination
6	Castors/floor gildes
7	Clamping lever height adjustment phlebotomy armrest
8	Clamping lever directional adjustment armpad (only with Vena <sup>®</sup> Multi-adjustable armrests)

### 2.1 Options and accessories

The Vena<sup>®</sup> Hydro can be configured with a choice of upholstery, legs/feet and armrests:

- › **Upholstery (1) (2), choose from:**
  - Stamskin Top
  - PUxx
- › **Castors/floor gildes (6)**
  - Protective floor glides
  - Hospital grade swivel castors with brake
- › **Armrests (4), choose from:**

**Vena® Fixed**  
 Vena® Fixed  
 Set of two armrests with fixed angle and PU arm pad  
 - art. no. 9080.00000

**Vena® Multi-adjustable PU**  
 Set of two multi-adjustable armrests with PU arm pad  
 - art. no. 9081.00000  
 - art. no. 9081.00000

**Vena® Multi-adjustable Stamskin/PUxx**  
 Set of two multi-adjustable armrests with upholstered arm pad  
 - art. n0. 9082.00000  
 armrest Stamskin Top  
 - art. no. 9083.00000  
 armrest PUxx

**The following accessories are optional extras with your product:**

- › **Get-up-brackets:** handles to support the patient when getting in and out of the chair.
- › **Foot-operated TB:** foot pedal for operating the seating angle of the chair and - if required - for placing the patient in the Trendelenburg position. If the Foot-operated TB pedal is chosen as an optional extra, it is still possible to adjust the seating angle of the patient chair using the 'Trendelenburg position operating lever' (5).

	<p><b>CAUTION: a Vena® armrest has its own label</b></p> <p>The Vena® armrests have their own CE product labels with individual UDI-DIs. The Vena® armrests are separate products and are interchangeable on the Hydrous Score Vena® phlebotomy chairs. Always consult the user manual of the Vena® Line phlebotomy chair and the item numbers on the labels of the relevant Vena® phlebotomy chairs and Vena® armrests.</p>
	<p><b>CAUTION: only use accessories intended for the product</b></p> <p>The use of any accessories that have not been approved for the intended use by the manufacturer is considered incorrect use and will invalidate the strict liability of the manufacturer for the use of the Vena Hydro for the duration of the relevant non-approved configuration usage.</p>
	<p><b>WARNING: always install two Vena® armrests on a Vena® Hydro treatment chair</b></p> <p>The armrest prevents persons from falling sideways from the chair.</p>
	<p><b>WARNING: use only manufacturer-approved armrests on the Vena® Hydro treatment chair</b></p> <p>Armrests not tested by the manufacturer in combination with the Vena® Hydro must not be attached to the Vena® Hydro.</p>

### 3. Vena® Hydro installation and first use

#### 3.1 Unpacking the product

The product is supplied fully assembled, in a box and on a pallet. The packaging must be undamaged upon receipt. If the packaging is found to be damaged, notify the carrier and immediately notify the Vena® Hydro supplier. If possible, add photos of the damaged packaging and the shipping label attached to the packaging.

Any undamaged packaging can be removed and disposed of. The pallet is made from wood and can be recycled; the cardboard should be disposed of as paper; the plastic can be disposed of as plastic.

#### 3.2 Installation of the product

Check that the product was supplied correctly, with the correct accessories and options, and inspect the product to ensure that it was supplied correctly and undamaged. In the event of damage, incomplete or incorrect delivery please contact the supplier immediately and include a photo of the damaged components and a photo of the product label (CE label).

Clean the product prior to placing it in the treatment room. Consult chapter 5 "Instructions for daily maintenance and cleaning"

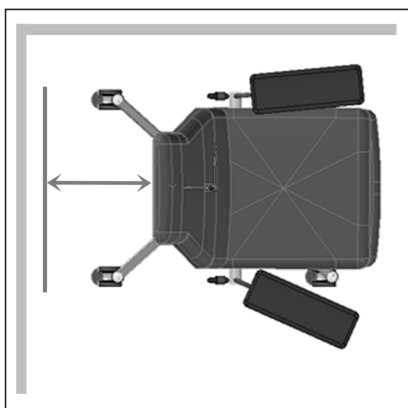
For correct placement in the room, follow the instructions in chapter 4.1 "Preparing for treatment".

Take note of this user manual for correct use of the Vena<sup>®</sup> Hydro

## 4. Instructions for use

### 4.1 Preparing for treatment

Carry out the following actions prior to the patient being seated in the Vena Hydro patient chair:


**1. Make sure the chair is placed in free space, unobstructed in the room.**

Before a patient is called in for blood collection, check that the Vena Hydro patient chair is in unobstructed in the room, making sure there is sufficient space so that the chair does not touch anything in its surroundings when placed in a reclining position.

**2. The patient chair must be on a solid surface**

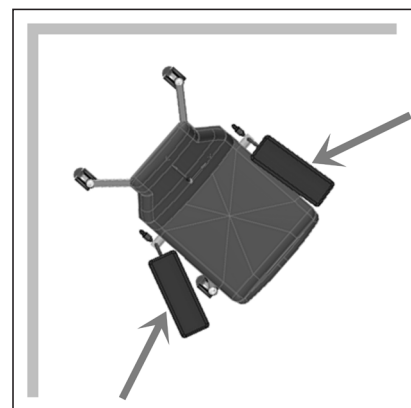If castors were selected for "(6) Castors/floor gildes (6)": lock all 5 wheels by pushing the brake clips down.

**3. Check that the patient chair is fitted with two blood collection armrests**

The chair must be fitted with two armrests, placed in the designated armrest slots at the side of the seat.

**4. Check the upholstery**

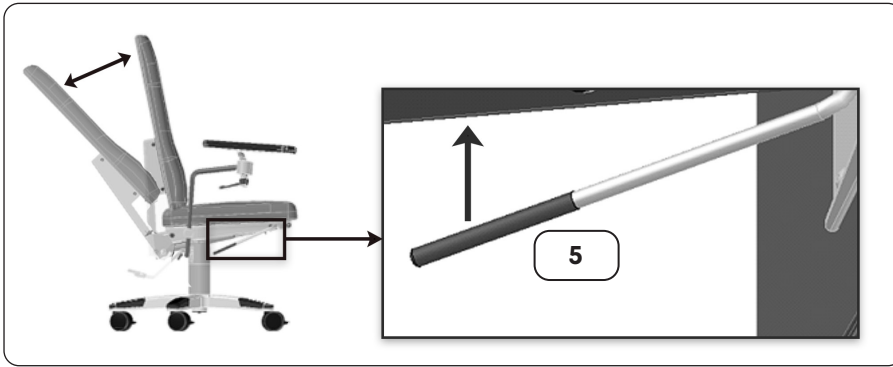
Ensure there is no damage to the upholstery. Check there are no rips or tears and ensure that the stitching is intact.

**5. Clean the chair**

Follow the instructions in chapter 5 "Instructions for daily maintenance and cleaning".

	<p><b>WARNING: Patient chair must be braked before use.</b> A patient must take place in a braked chair only. The chair with castors much have its brakes applied during treatment.</p>
	<p><b>WARNING: Patient chair must be fitted with two armrests</b> An armrest must be in place on both sides during treatment to prevent the patient to fall sideways from the chair if they become unwell.</p>
	<p><b>CAUTION: chair must be positioned in unobstructed space in the room</b> Ensure the chair is installed in free, unobstructed space and does not come in contact with the walls and/or other objects in the room to prevent damage to the patient chair or other objects.</p>
	<p><b>CAUTION: upholstery must be undamaged</b> Damaged upholstery may lead to accumulation of dirt and bacterial growth from within. Damaged upholstered components must be repaired. Contact the supplier or manufacturer of the device.</p>

### 4.2 Adjusting the angle of the backrest and legrest

Use operating lever (5) underneath the seat to adjust the angle of the back rest. This lever is located under the seat on both the left and right sides of the chair. The position of the backrest can be adjusted by pulling the lever upwards and pushing the backrest down.

The backrest is infinitely adjustable using operating lever (5). Releasing the lever locks the adjustment automatically.

Pulling the control lever up and pushing the backrest up simultaneously, brings the chair back to an upright position.

	<p><b>WARNING: patients are not permitted to operate the Vena® Hydro treatment chair.</b> Only persons familiar with the operation of the chair and the user manual are permitted to operate the Vena Hydro treatment chair.</p>
	<p><b>CAUTION: Always use two hands to adjust the chair.</b> Use one hand to operate the lever and use the other hand to adjust the seat's backrest. Do not pull the leg support.</p>

### 4.3 Getting in and out of the Vena® Hydro

Invite the patient to take place in the chair. The patient may only be seated in the Vena® Hydro patient chair when it is in an upright position. When getting out of the chair, it must also be returned to the chair's most upright position. Do this by pulling lever 5 upwards and guiding the chair's backrest into the most upright position. A patient must take place in the chair in an upright position and the patient must properly position themselves with their back against the backrest.

For additional ease and comfort in getting in and out of the Vena® Hydro, optional stand-assist handles can be fitted to the Vena® Hydro chair.

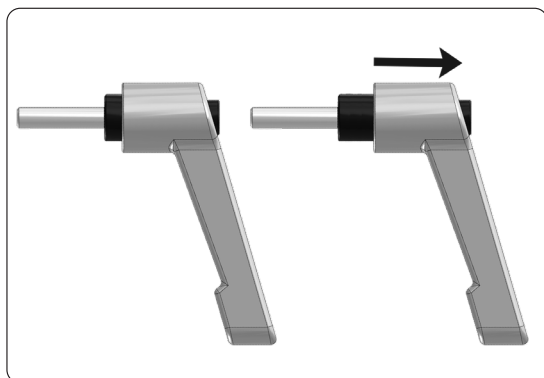
	<p><b>WARNING: when the chair is in the reclining position, a person may be hindered by armrests or stand-assist handles when getting in and out of the chair</b> A person must only sit down in or get out of a chair in the most upright seating position.</p>
	<p><b>CAUTION: the chair must be in a fully upright sitting position when getting in and out of the chair</b></p>

#### 4.4 Operating the clamping lever

The Vena<sup>®</sup> Hydro armrests can be adjusted to allow you to position the patient's arm properly for inserting a needle into the vein. A good arm position means the operator can adopt a better work posture without tension when injecting the patient. Use the locking clamping lever (7) (8) to properly secure and easily release the armrest or the arm pad.

The advantage of a clamping is that the handle can also rotate freely, which means you can set the handle in a way that allows you to grab or exert force easily when adjusting the armrest.

7  
8


Adjusting the position and hold of the clamping lever:

1. Hold the handle in one hand.
2. While holding the handle, pull it backwards.
3. When pulling the handle the other way, it can be rotated freely.
4. Adjust the handle to a position that works for you, the operator.
5. Release the handle.

This should improve the position of the handle. As a result, you have a better hold on the clamping lever, making it easier to secure the armrest. Repeat the steps above to adjust the position of the handle to where you can exert enough force to operate the clamping lever.


#### 4.5 Adjusting the Armrest

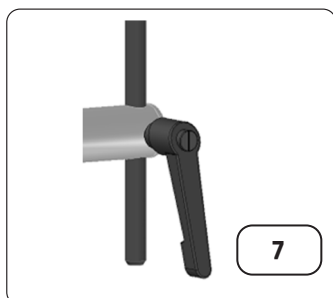
drawing 1

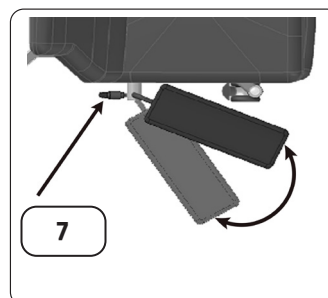drawing 2

drawing 3

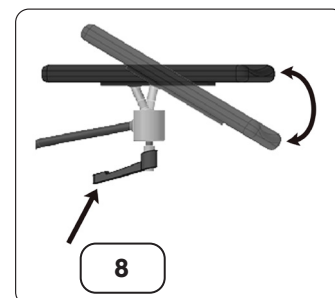drawing 4

##### Height and depth adjustment (drawings 1 to 3)

Use clamping lever (7) to adjust the height of the armrest.

- Turn the handle on drawing 1 anticlockwise to release the armrest.
- Set the armrest to the correct working height (drawing 2).
- Rotate the armrest outward (drawing 3) or inward for a comfortable work posture.
- Secure the armrest by rotating the clamping lever handle clockwise.

##### Adjusting the arm pad (drawing 4)

(only applicable to the Vena<sup>®</sup> Multi-Adjustable armrests with item nos. 9081.00000, 9082.00000 and 9083.00000)

- Release the clamping lever (8) by rotating it anticlockwise.
- Position the angle of the arm pad.
- Secure clamping lever 8 to secure the arm pad by rotating the clamping lever handle clockwise.

Start the treatment once all clamping levers are properly secured and the armrests are locked in.

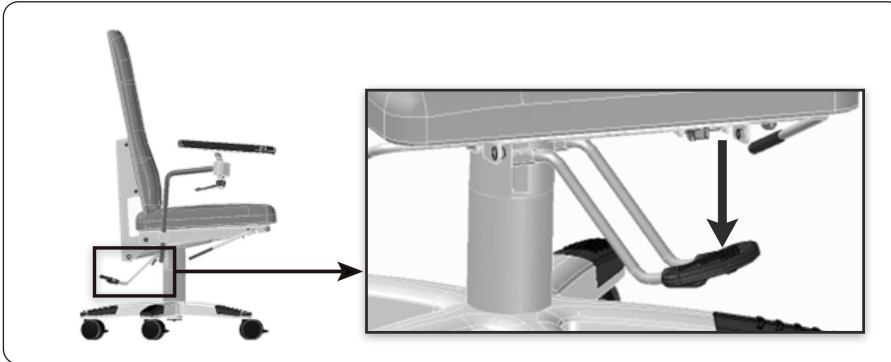
**WARNING: the Vena<sup>®</sup> armrests must be secured during treatment**

If an armrest is not properly secured, it may turn away or slide down during treatment.

#### 4.6 Foot pedal for height adjustment and rotation lock

The chair can be adjusted in height using the foot pedal. Use your foot to operate the pedal.

**Adjusting chair height:** by placing the foot on the pedal, you depress the pedal. By pressing and pumping half way (i.e. not pushing through to the lowest position), you pump the seat height up.

Pushing the pedal all the way to the lowest position and holding it will lower the seat height.

**Seat rotation lock:** pushing the foot pedal up with the foot will block the rotation of the chair. Pressing the foot pedal down again allows the chair to rotate on its axis again.

#### 4.7 Fainting of the patient

During blood collection, donation or injection, the patient may be at risk for vasovagal syncope (fainting) as a reaction to seeing blood and/or needles. Ensure the safety of the patient by following the protocols in place on site. Because the patient may fall over during vasovagal syncope, the Vena Hydro should always be equipped with two Score Vena Line armrests (see section 2.1) or MPC armrests (additionally available) on both sides of the chair.

	<p><b>WARNING: the Vena® Hydro must always be fitted with two Vena armrests</b> The patient may fall sideways when in a reclined position or if they become unwell. The armrests offer protection.</p>
	<p><b>WARNING: Always follow the institution's protocols when a patient becomes unwell</b> The Vena Hydro offers support when a patient becomes unwell. By being aware of and adhering to the protocols, you will know exactly which steps must be taken in the event of an emergency situation.</p>
	<p><b>WARNING: secure the needle first before placing the patient in the Trendelenburg position</b> Follow the institution's relevant protocols when a patient becomes unwell and faints (or is likely to).</p>

#### 4.8 Safety measures for the Vena® Hydro

	<p><b>WARNING: Do not use the backrest (1) as a seat</b> The chair may topple if the backrest is used as a seat.</p>
--	--

	<b>WARNING: do not lean on the armrests</b> The phlebotomy armrests are intended to support the arm during blood collection.
	<b>WARNING: armrests are not stand-assist supports</b> To sit down or stand up, a patient can use stand-assist handles (optional extras) to support them.
	<b>WARNING: do not use the Vena<sup>®</sup> Hydro for patient transport</b> The Vena <sup>®</sup> Hydro is not suitable for transporting patients.
	<b>CAUTION: castors must only be used to move the chair without a patient</b> If the Vena <sup>®</sup> Hydro is regularly moved within the room or between rooms, there is an option to choose castors as legs/feet (6). These are available to order as optional extras.

## 5. Instructions for daily maintenance and cleaning

The Vena Hydro is a non-sterile product. For daily cleaning, wipe the Vena Hydro with a damp cloth and use a non-alkaline detergent if needed.

### 5.1 Cleaning PU surfaces, chrome-plated and coated components, feet/castors

- › clean with a damp cloth, use a mild, non-abrasive detergent if needed.
- › Dry the Vena<sup>®</sup> Hydro using a dry cloth after using a damp cloth.
- › If the Vena<sup>®</sup> Hydro is fitted with castors, it must not be moved when the floor or castors are wet.
- › The gas spring fitted underneath the chair is used to adjust the chair. It should never be cleaned using water or any detergent.

In the event of contact with blood or other bodily fluids:

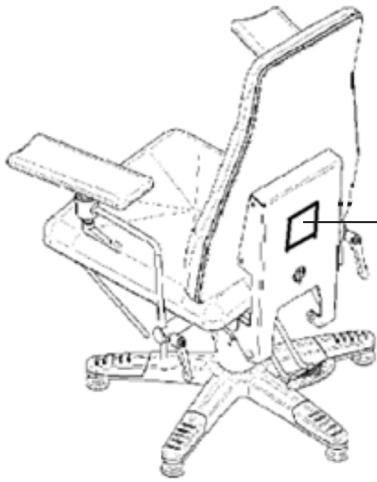
- › Clean PUR, chrome-plated, and coated components using a cold solution of 10,000 ppm chlorine and water. Wipe immediately using a clean, damp cloth. Finally, dry using a clean, dry cloth.

	<b>CAUTION: Clean the entire product before first use</b> Follow the relevant hygiene protocols for the work environment in which the Vena Hydro is used.
--	--

### 5.2 Cleaning the upholstery

The upholstered parts of the chair have been carefully selected to ensure their properties comply with the requirements of a medical environment. The Vena Hydro is supplied with Stamskin Top<sup>®</sup> or PUxx<sup>®</sup> upholstery as standard. The product label on the medical device states whether your Vena Hydro has Stamskin Top<sup>®</sup> or PUxx<sup>®</sup> upholstery.

<p><b>Vena Hydro</b> — <b>Stamskin Top - K85</b></p> <p>Phlebotomy Chair - Bloedafnamestoel</p> <p>Medical Device / Medisch Hulpmiddel</p> <p>Art.no.: 9200.10000.1K859081</p> <p><b>Score<sup>®</sup></b></p> <p>Score B.V. Feithspark 5 9356 BX Tolbert The Netherlands www.scoreseating.com</p>	
--	---

**CAUTION: identify the chair's upholstery for correct cleaning**

The product label on the back of the Vena Hydro provides information on the type of upholstery on your product. The cleaning instructions for Stamskin Top and PUxx differ. Make sure you check the type of upholstery on your chair beforehand and familiarise yourself with the cleaning method for this upholstery.

### 5.2.1 Stamskin Top upholstery: properties and cleaning method

**Stamskin Top properties:**

- Multi-layered artificial leather PVC on a polyamide-jersey base cloth.
- Resistant to urine, blood, sweat, mould, and water.
- Abrasion resistance > 120,000 Martindale (EN ISO 5470-2)
- Fire-resistance: M2/NFP92.507, Test cigarette / Match EN 1021.1-1021.2, IMO MSC.307 (88) (201 FTP Code) Annex 1 part 8, Aeronautical test FAR/JAR 28-853/855.
- Fungicidal: Degree 0, excellent (EN ISO 846-A).
- Sanitized® hygiene function: protection against mould and algae formation.

**Stamskin Top cleaning method:**

- › Wipe the Stamskin Top upholstery daily using a damp lukewarm/warm cloth. If needed, use a mild detergent (Ph-neutral, free from acidic or alkaline excipients).
- › Do not use alcohol-based, chlorine-based, ammonia-based or acid-based detergents for regular maintenance. Do not use solvents or abrasives.
- › After cleaning, dry the upholstery using a dry cloth.
- › For general stains, we recommend Stamskin Cleaner. Apply the Stamskin Cleaner using a soft brush and use circular motions on the surface to be treated. Leave the Stamskin Cleaner to penetrate for 3 minutes. Then remove the Stamskin Cleaner, together with the dirt, using a dry cloth.
- › In the event blood or other bodily fluids come into contact with the Stamskin Top upholstery, disinfect immediately using a cold solution of 10,000 ppm chlorine and water. After applying, wipe using a clean, damp cloth and dry using a clean, dry cloth.

A list of detergents tested on Stamskin Top is available. Please contact your product's supplier.

**WARNING: alcohol, chlorine, ammonia or acid-based detergents may damage Stamskin Top upholstery**  
If these substances come into contact with Stamskin Top upholstery, the residue must be wiped off immediately using a damp cloth. Follow the instructions.

## 5.2.2 PUxx upholstery: properties and cleaning method

### PUxx1 properties:

- 100% polyurethane on polyester/cotton backing. Completely free from PVC and phthalates.
- 100% waterproof, resistant to urine and blood.
- Abrasion resistance 300,000 Martindale (EN ISO 5470-2)
- Fire-resistance: UK BS5852 Crib 5, EN 1021 1-2, CALL117-2013, IMO 2010 FTP: part 8
- Light-fastness 6-7 (8 is very good)
- Can be disinfected using alcohol, chlorine, or hydrogen peroxide solutions
- Oeko-Tex certified

### PUxx cleaning method:

- › Wipe PUxx daily using a damp lukewarm/warm cloth. If needed, use a detergent. PUxx is resistant to most detergents and disinfectants made from 70% alcohol, chlorine, or ammonia. Follow the instructions given by the manufacturer of the disinfectant. Detergents and disinfectants must always be wiped off using a clean, damp cloth. Do not use solvents or abrasives.
- › After cleaning, dry the upholstery using a dry cloth.
- › The PUxx components that come into contact with skin (such as the upholstered armrests) should be periodically cleaned using plenty of warm water and Nirocleaner or any other alkaline detergent to remove residual soap and skin oils. Use a brush if needed. Allow these detergents to penetrate for 3 minutes (preferable) before removing the detergent and dirt using a clean, damp cloth. Thorough removal of the detergent will slow down the build up of dirt.
- › Remove bloodstains immediately using a cold disinfectant.

## 5.3 Cleaning after a blood squirt incident

In the event of a squirt incident where the Vena<sup>®</sup> Hydro has come into contact with blood, the site of the incident must be cleaned immediately using a permitted disinfectant. Heat fixates bloodstains. As such, a cold detergent or disinfectant must always be used.

# 6. Maintenance

Periodically check that the product is without damage and functions properly. Periodic inspections contribute to a longer service life and problem-free usage. Reduced functionality may signal future issues. If the chair is damaged or does not function as it should, please contact your supplier. In case of doubt, contact the supplier as listed on page 2 "contact details".

General maintenance can be carried out by the medical technical department or by the supplier of your Vena<sup>®</sup> Hydro. For repairs and/or parts, contact your product's supplier.

	<p><b>WARNING: do not use a defective product.</b></p> <p>Any defect may result in injury or discomfort to the patient or the operator of the Vena Hydro. If a defect is found, please discontinue use of the treatment chair immediately and repair the damage. Contact the technical department or the supplier of the product for maintenance.</p>
	<p><b>CAUTION: report unusual wear and tear immediately</b></p> <p>If the product or a component thereof experiences visible wear and tear, contact the product's supplier so that the wear and tear can be identified in a timely manner for safe future usage. In case of doubt, contact the supplier.</p>

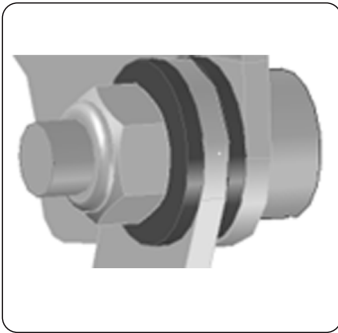
## 6.1 Maintenance interval

The Vena Hydro was designed and tested for a normal service life of 10 years. There is a five-year guarantee on the product. The product must bear the product label with CE logo. This label lists the UDI-DI, the serial number and the production date. If the product has no label, the guarantee will be nil and void. In case of damage or erroneous removal of the label, please contact the supplier or manufacturer of the device immediately.

Inspect the entire product at least once a year for functioning and damage. This check can be carried out by the on-site medical technical department or by the product's supplier. Individual components and functionalities can be checked more frequently if desired. Damaged components must be replaced.

	<p><b>CAUTION: inspect the product for wear and tear or defects at least once a year</b></p> <p>Periodic checks contribute to the safety of the medical device.</p>
---	---

## 6.2 Regulatory checks

### Fittings and mounting materials

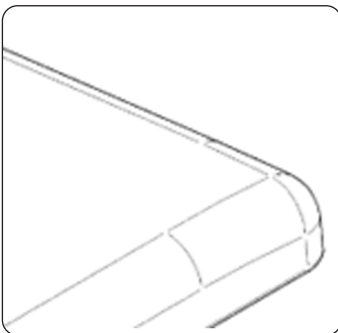
Check the fittings and mounting materials (underneath the chair) and inspect for loose parts and unusual wear and tear. Unusual wear and tear may lead to future issues. Always contact your supplier in case of doubt.

### Clamping levers

Check the functioning of the locking grips. Check that the locking grips are still rotating properly in the thread and that the locking function is still in order.

Damages locking grips must be replaced. For this, contact the supplier.

### Upholstery

Check the upholstery for damage. Pay attention to the stitched seams. The chapter on safety instructions ... lists the impermissible damages.

Product components that display such damage must be replaced immediately before the product can be used again.

### Gas spring

Check the functioning of the product by fully adjusting it to the Trendelenburg position. The gas spring should move an unloaded chair into the neutral position unaided.

A leaking gas spring must be replaced. For a replacement gas spring, contact the supplier of the product.

### Floor gliders / castors

The Vena Hydro is fitted with five floor glides as standard. Brake loaded castors are optionally available. Check that all the floor glides, or castors, are present and that they all sit on the surface properly. Inspect the floor glides or castors for any damage.

Check each castor for dirt accumulation, which could affect its movement. Check that the castor can rotate and roll freely, and check correct functioning of the brake. If needed, clean the castors using clean and dry compressed air.

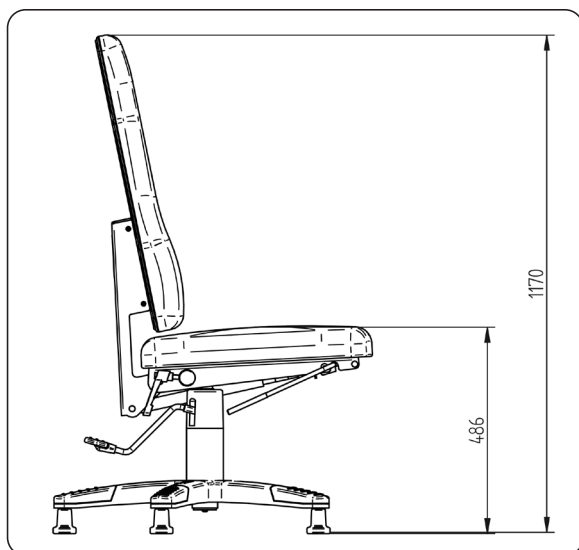
In case of defects, the feet or castors can be replaced. Contact the supplier for replacement parts.

### **CAUTION: contact the supplier if the product is not functioning properly.**

Always ensure the product functions well. Contact the supplier or manufacturer of the device if you are uncertain about your product's functionalities.

## 7. Dimensions Vena Hydro

- back angle adjustment: 30° degrees backwards
- height adjustment: 486 - 656 mm (adjustment range 17 cm) by means of hydraulic foot pump
- Ø star base: 700 mm
- dimensions chair (width x length ): 610mm x 600mm
- seat dimensions (width x length ): 540 mm x 530 mm

## 8. Complaints and safety notices

Score B.V. has taken utmost care to ensure the safety and functionality of this medical device.

### 8.1 Product safety and functionality notifications.

Despite our great dedication, it may happen that the product does not function properly or that a dangerous situation has arisen that was not mentioned in this user manual. This does not automatically result in a complaint, although the findings could result in future complaints.

Score B.V. believes it is very important to receive information about the safety and the functioning of its products. If a situation has occurred that could cast doubt on the safety of a patient or other user, or if the Vena Hydro is not performing as it should, please notify the supplier of the product or manufacturer directly as soon as possible. To ensure appropriate follow-up of the notification, please include a photo of the relevant product's CE product label.

### 8.2 Complaints

In the event of a complaint about the product, please notify the supplier of the Vena Hydro directly, or the manufacturer of the product. We will ask for:

- the UDI-DI and the serial number of the product. These are listed on the product's CE label
- description of the complaint
- any images to support the complaint or the chair's defect.

### 8.3 Contact details

The supplier's and/or manufacturer's contact details are listed the back page of this manual.

## 9. Quick guide

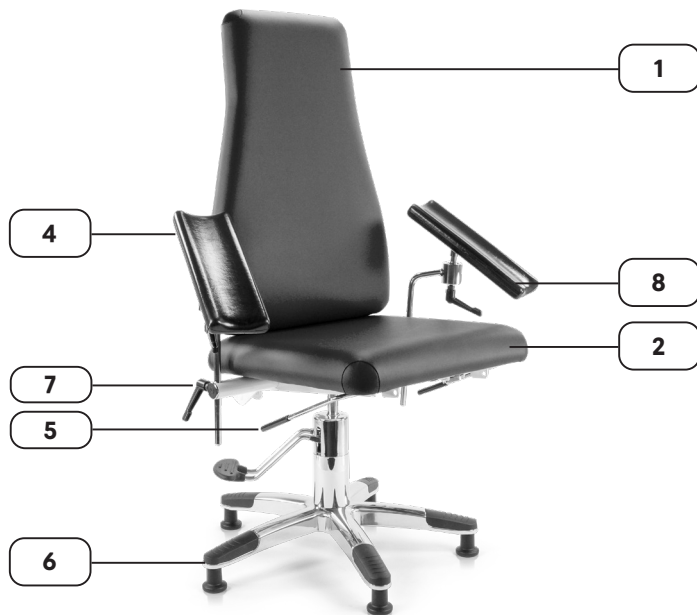
For daily use, an operator may use the quick guide. The quick guide can be found in this chapter. The quick guide is derived from this user manual and does not replace this user manual. If the quick guide is distributed, always list the location of this full manual as well.

**CAUTION: always store the user manual in the vicinity of the Vena Hydro.**  
Ensure that people operating the Vena Hydro are aware of the full manual's location.


## Vena Hydro - Quick guide

This quick guide acts as a support tool Before working with the Vena Hydro, you must be familiar with the contents of the full user manual.


Number	Component
1	Backrest
2	Seat
3	Foot pedal for height adjustment
4	Armrest for blood collection
5	Operating lever for adjusting the backrest inclination
6	Castors/floor gildes
7	Clamping lever height adjustment phlebotomy armrest
8	Clamping lever directional adjustment armpad (only with Vena <sup>®</sup> Multi-adjustable armrests)

### Adjusting the backrest inclination



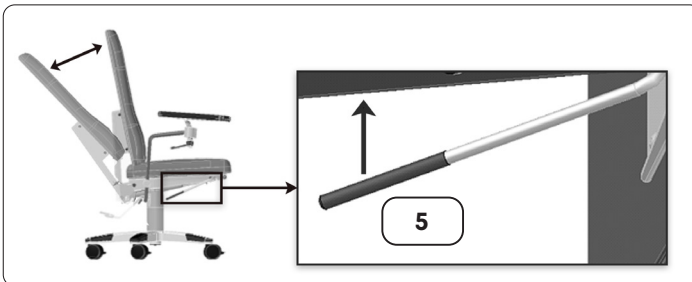
### Foot pedal for height adjustment

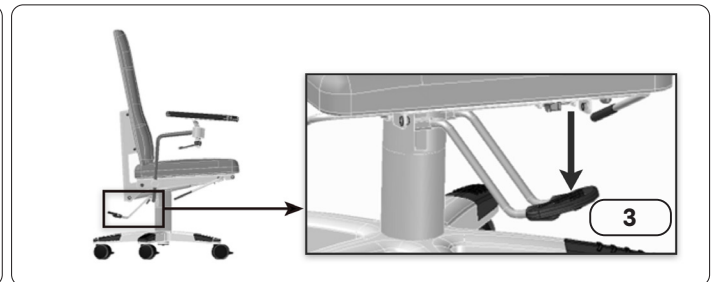
Up: push pedal halfway down and pump;  
Down: push all the way down;  
Seat rotation lock: push pedal up.

### Adjusting the height depth of the armrest:

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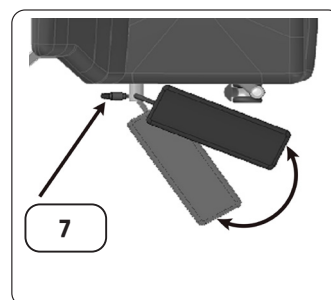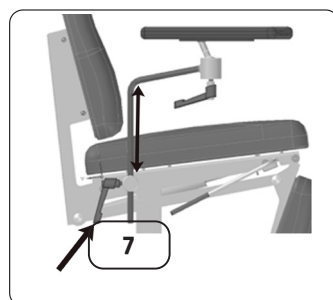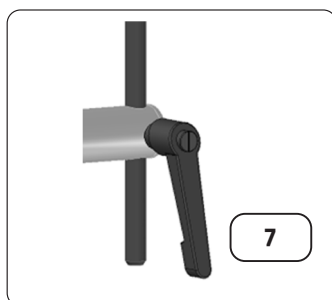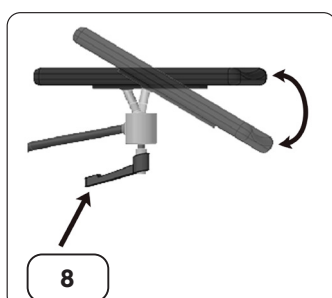The diagram shows a rectangular identification label for the Vena Hydro device. Callout 1 points to the product name and type: "Vena Hydro - Stamskin Top - K85". Callout 2 points to the article number: "Art.no.: 9200.10000.1K859081". Callout 3 points to a QR code and a list of UDI-DI components: (01) 08717228183881, (11) 230830, (10) 1969094, and (21) 2315520/001. The label also contains the Score logo, company address (Score B.V., Feithspark 5, 9356 BX Tolbert, The Netherlands, www.scoreseating.com), CE mark, and serial/production date information (SERIAL: 2315520/001, PROD DATE: 30-8-2023). A vertical code "BPS06-77-A" is located on the right side.

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The diagram shows a rectangular label for the 'Vena Hydro' device. Callout 1 points to the product name and color code: 'Vena Hydro - Stamskin Top - K85'. Callout 2 points to the article number: 'Art.no.: 9200.10000.1K859081'. Callout 3 points to a QR code and a list of UDI-DI components: (01) 08717228183881, (11) 230830, (10) 1969094, and (21) 2315520/001. The label also features the Score logo, CE mark, company address (Score B.V., Feithspark 5, 9356 BX Tolbert, The Netherlands), website (www.scoreseating.com), and serial/production dates (SERIAL: 2315520/001, PROD DATE: 30-8-2023). A vertical code 'BPS06-77-A' is on the right side.

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